

THE LEADERS IN RECREATION, FITNESS AND ABILITY!

9889 Garrymore Ln Missoula, MT 59808 888-687-3552 | +1-406-549-0769 www.aquacreek.com

2nd Battery Instructions

PART #: F-004AB

MANDATORY LEAVE THIS MANUAL WITH LIFT OWNER

Read and follow all instructions. Lift safety can only be ensured if the lift is installed and operated according to these instructions.

- NEVER permit children to play on or around the lift
- Do not allow children to use the lift without adult supervision
- NEVER apply direct water pressure to the electronics
- NEVER use the lift with a dry pool

READ CAREFULLY

Check entire box and all packing materials for parts. Before beginning assembly, read the instructions and identify parts using the figures and parts listed in this document.


It is critical that all parts be carefully inspected prior to installation. If any damage occurred in transit, Aqua Creek Products, LLC must be notified within three days of receipt of unit.

Proper installation cannot be overstressed, as an improper installation voids Aqua Creek's warranty and may affect the safety of the user.



F-004AB: 2ND Battery Instructions

The lift comes with one 24V Battery and a wall charger unit. This kit includes a 2nd 24V Battery, which allows one battery to be on the unit while the other is on the charger. To take full advantage of the 2nd battery; one battery should always be kept on the charger. This should ensure that if the battery on the lift becomes depleted during use, a charged battery is always available.

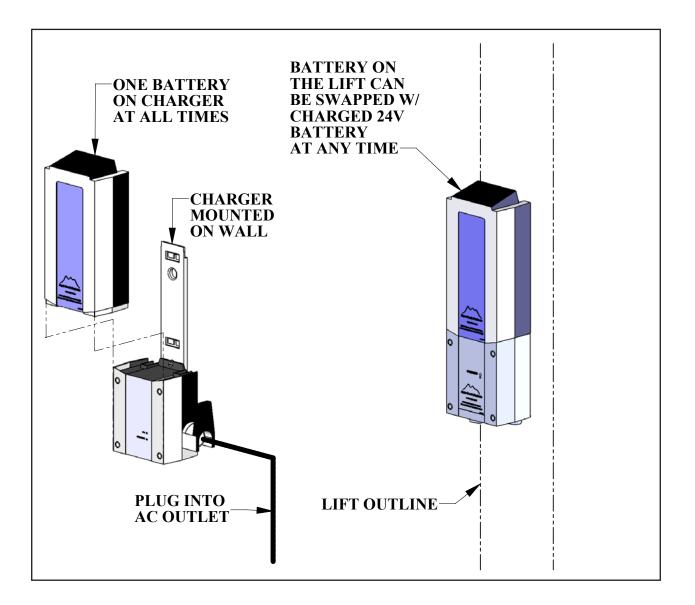

To recharge the battery, simply grip the top of the battery and depress the clip on the back of the battery. This will unclip the battery from the control box bracket and the battery can then be clipped into the wall mount charger. When the <u>charger</u> is plugged in the green **ON** light will illuminate. When the <u>battery</u> is correctly mounted to the charger, the orange **CHARGE** light will illuminate. When the battery is fully charged, the **CHARGE** light will **GO OFF** and the battery is ready to use. The battery charger will automatically shut off when the battery is fully charged, so the battery can stay on the charger any time it is not being used.

2

Battery Maintenance

- 1) <u>**RECHARGE THE BATTERY AS OFTEN AS POSSIBLE.</u>** The battery power unit is a sealed lead acid battery pack. Frequent recharges will prolong the battery life and maintain a high cold amperage capacity.</u>
- 2) Do NOT expose the battery to freezing temperatures.
- 3) Do NOT expose the battery to prolonged periods of extreme heat or severe cold temperatures.
- 4) Do NOT expose the battery to long periods of direct sunlight. Heat will shorten your battery life. Optimum operating temperature is below 20 degrees Celsius (68 degrees F.)
- 5) Never drop or bump the battery pack. This may result in loose or damaged internal connections, and may bend the battery terminals.
- 6) Keep battery terminals clean and free from dirt and corrosion to ensure good contact. The terminals may be cleaned using Scotch Brite or any similar product. Clean off any corrosion or discoloration on the very top surfaces of the battery contacts and then apply dielectric grease (supplied with lift) as shown in the figure below.

